MARQUE: SPORT-ELEC

REFERENCE: FREE ACTION

CODIC: 3856089

# FREE ACTION

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**EN** You have just purchased an advanced and powerful professional device. You should take the time to familiarise yourself with how this device operates by reading these instructions carefully.

**IMPORTANT > Recharge the device completely prior to initial use.**

**EMERGENCY STOP:** Press the "ON/OFF" button (⏻) located on each motor.

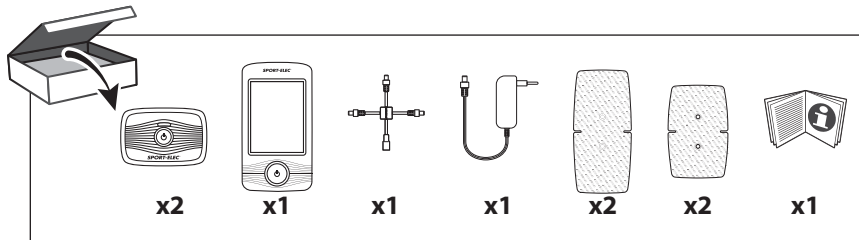
#### THERAPEUTIC APPLICATIONS

- ▶ To strengthen and/or to maintain muscle strength.
- ▶ To ease muscular tensions.
- ▶ To prevent muscular atrophy and to treat pain.

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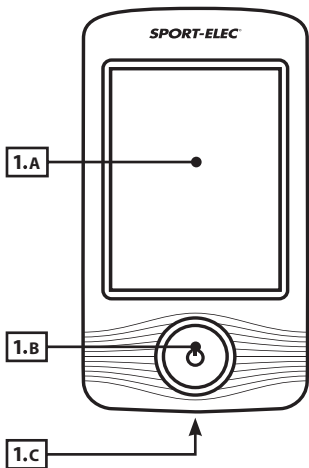
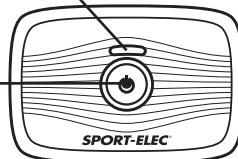
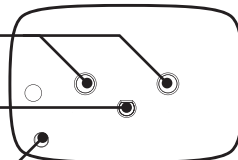
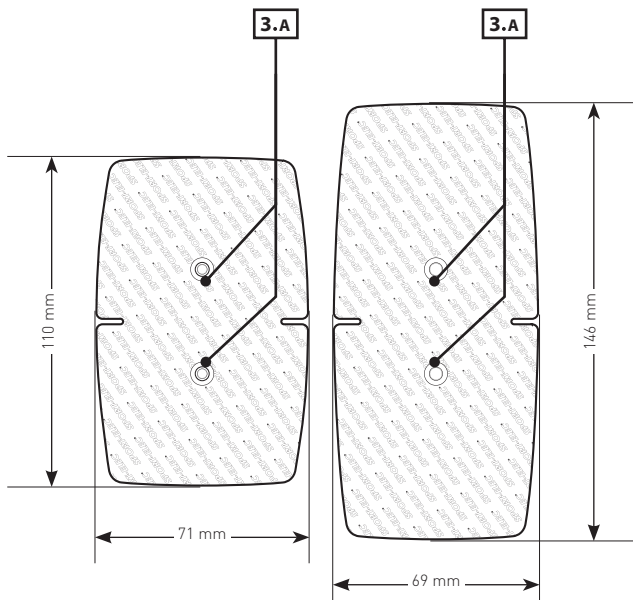
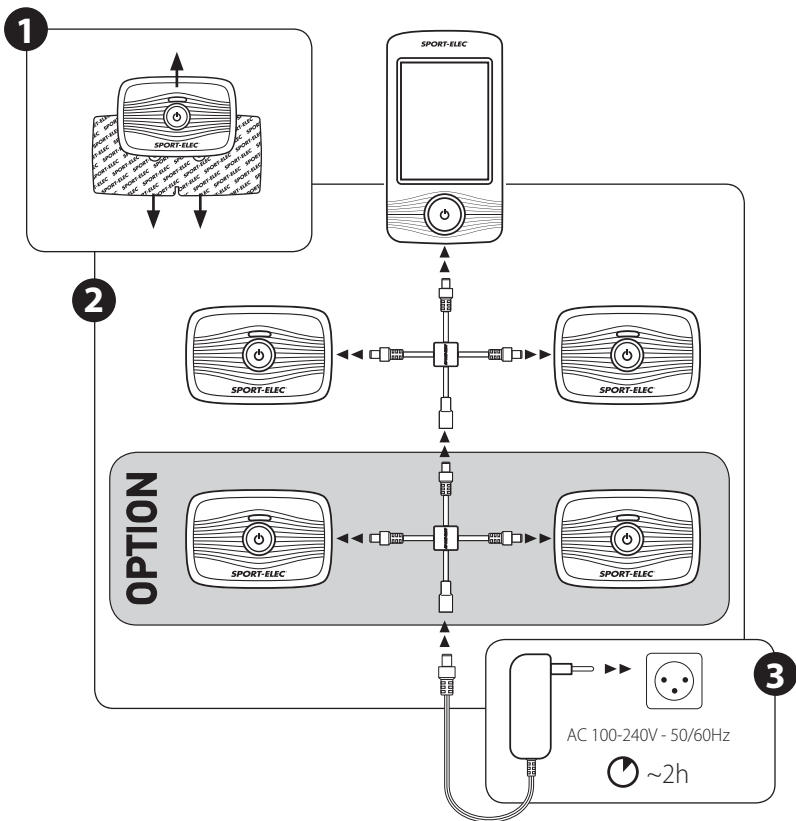
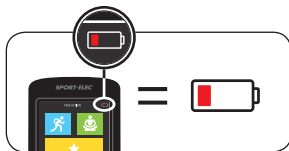
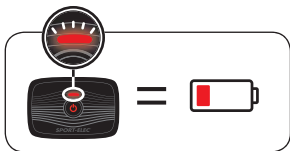
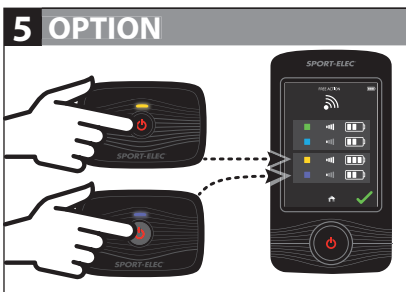
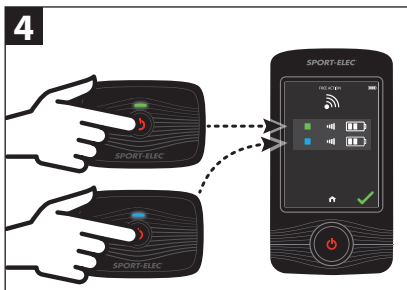
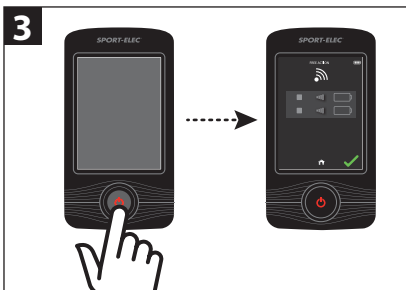
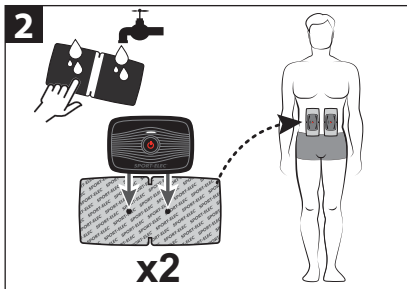
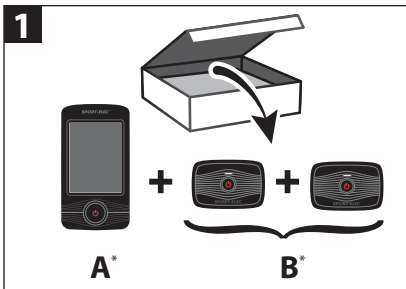
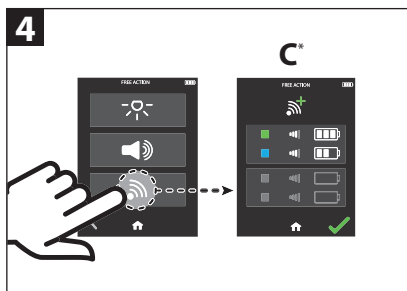
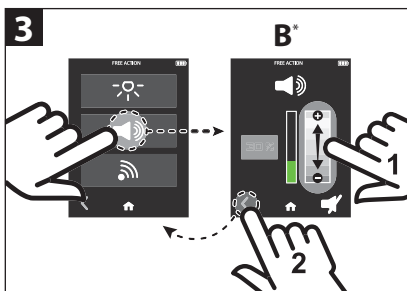
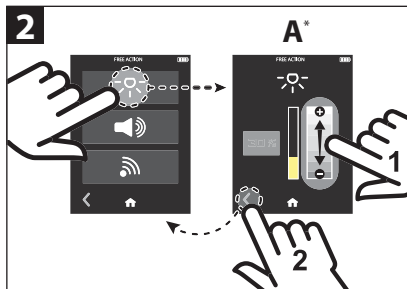
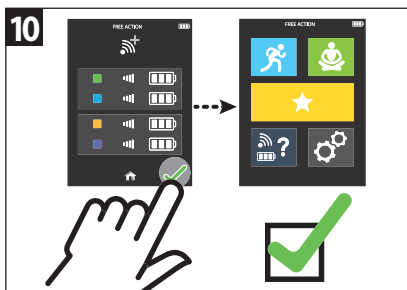
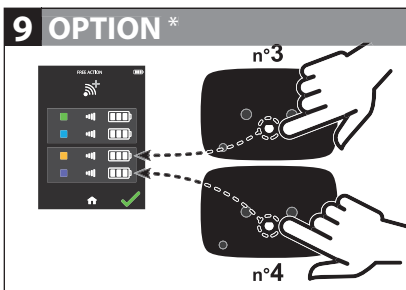
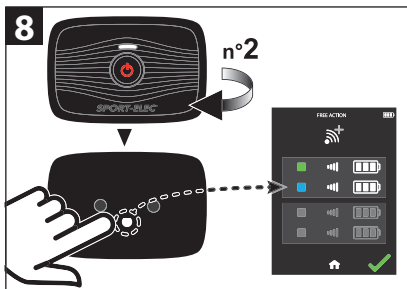
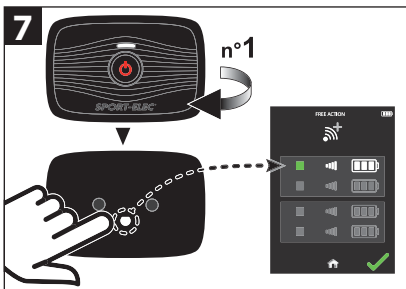
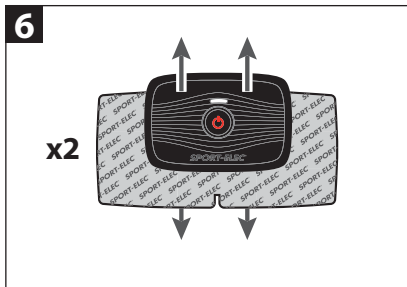
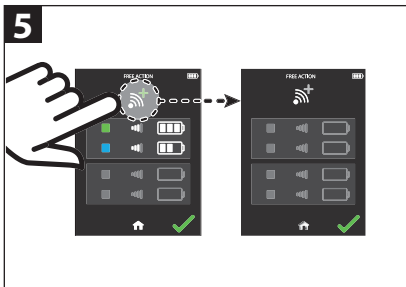
**EN** The "CONNECTING MOTORS" function should only be used:

- ▶ if one of the motors is not recognised by the remote,
- ▶ if you wish to add one or several motor(s),
- ▶ or if recommended by the After-sales service.

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### EN SERVICING & CLEANING RECOMMENDATIONS

#### ► Terminals:

The terminals can be cleaned with a cloth soaked in disinfectant (such as Mercryl) diluted with water to 1/10.

#### ► Adhesive electrodes:

Rehydrate before and after use.

If necessary, use disinfectant (e.g.: Hexomedine).

Leave on the original support base, in a plastic bag at between 5°C and 10°C.

### CONDITIONS FOR USE

- Temperature: 5°C - 40°C (max. 35°C while charging)
- Relative humidity: 20% - 65%

### STORAGE CONDITIONS

- Storage temperature: 0°C - 45°C
- Relative humidity in the storage area: 10% - 90%
- Keep in a dry location, away from direct sunlight.

### RECOMMENDATIONS:

Make sure you comply with national regulations

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EN SOLVING PROBLEMS | DE PROBLEMLÖSUNG | ES SOLUCIÓN DE PROBLEMAS  
IT RISOLUZIONE DEI PROBLEMI | NL PROBLEMEN OPLOSSEN | PT RESOLUÇÃO DE PROBLEMAS |  
CZ ŘEŠENÍ PROBLÉMU | HU PROBLÉMAMEGOLDÁS | PL ROZWIĄZYWANIE PROBLEMÓW |  
RO REZOLVAREA PROBLEMELOR | BG РАЗРЕШЕНИЕ НА ПРОБЛЕМИ | ES PROBLEMLÖSUNG |  
RU ПРОБЛЕМЫ И СПОСОБЫ ИХ РЕШЕНИЯ | TR SORUN GİDERME | حل المشاكل AR

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**EN** If the problem persists after carrying out the following checks, contact our After-sales service at +33 2 32 96 50 51 or sav@sport-elec.com

## MOTOR LED FLASHES RED

- ▶ Charge (see chapter III) and retry.

## TRIANGLE SYMBOL

### or MOTOR LED FLASHES WHITE

- ▶ Turn all terminals (remote control + motors) OFF and ON again
- ▶ If the problem persists, contact the after-sales service directly

## WHITE BARGRAPH

The electrode in question is disconnected.

- ▶ Try new electrodes.
- ▶ Check the contact between the electrode and the skin.
- ▶ Check that the motor is fixed to the electrode.

- ▶ Check that the motor is ON.
- ▶ Then try to increase power again.

## ONE OR SEVERAL TERMINALS FAIL TO COME ON OR EXPECTEDLY TURN OFF

- ▶ Charge all of the terminals for approximately 2 hours.
- ▶ Restart your terminals.
- ▶ Restart the preferred program.

## I CAN'T FEEL ANY MUSCLE CONTRACTIONS

- ▶ Try new electrodes.
- ▶ Check that the electrodes are in contact with the skin.
- ▶ Check that the motors are fixed to the electrodes.
- ▶ Check the connection between the motors and the

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**EN WARRANTY :** We accord a 24-month warranty from the date of purchase for the products we distribute. We will repair all manufacturing or material defects free of charge throughout the entire warranty period, however, we reserve the right to decide if some parts must be repaired or replaced, or if the device itself must be replaced. The warranty will not apply in the event of damage caused by inappropriate use (impact, use of direct current, incorrect voltage, etc.), normal usage will not cause prejudice to the satisfactory operation and value of the device. The warranty will expire if repairs are carried out by unauthorised individuals or if spare parts by another brand are used. The warranty will only be valid if the date of purchase and the stamp and signature of the store appear on the warranty card (or upon presentation of the receipt). No other forms of justification will be accepted for the warranty, unless legal provisions specify otherwise.

**ELECTRODE WARRANTY :** The adhesive electrodes are intended for strictly personal use. Electrodes may not be returned or exchanged for hygiene reasons.

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## GUIDANCE AND MANUFACTURER'S DECLARATION ELECTROMAGNETIC EMISSIONS

The Free Action is intended for use in the electromagnetic environment specified below. The customer or the user of the Free Action should assure that it is used in such an environment.

EMISSIONS	COMPLIANCE	ELECTROMAGNETIC ENVIRONMENT - GUIDANCE
RF emissions CISPR 11	Group 1	The Free Action uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.  The Free Action is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Conform Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Conform	

## GUIDANCE AND MANUFACTURER'S DECLARATION ELECTROMAGNETIC IMMUNITY

The Free Action is intended for use in the electromagnetic environment specified below. The customer or the user of the Free Action should assure that it is used in such an environment.

IMMUNITY TEST	IEC 60601 TEST LEVEL	COMPLIANCE LEVEL	ELECTROMAGNETIC ENVIRONMENT - GUIDANCE
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines and dc lines	±2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
	±1 kV for input/output lines	Not applicable for input/output lines	
Surge IEC 61000-4-5	±1 kV line(s) and neutral ± 2 kV line(s) to earth	±1 kV line(s) and neutral N/A(57.2 EN61000-4-5)	Mains power quality should be that of a typical commercial or hospital environment. If a dips or an interruption of mains power occurs, the current of the Free Action may be dropped off from normal level, it may be necessary to use uninterruptible power supply or a battery.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % U <sub>r</sub> (>95 % dip in U <sub>r</sub> ) for 0,5 cycle	<5 % U <sub>r</sub> (>95 % dip in U <sub>r</sub> ) for 0,5 cycle	
	40 % U <sub>r</sub> (60 % dip in U <sub>r</sub> ) for 5 cycles	40 % U <sub>r</sub> (60 % dip in U <sub>r</sub> ) for 5 cycles	
	70 % U <sub>r</sub> (30 % dip in U <sub>r</sub> ) for 25 cycles	70 % U <sub>r</sub> (30 % dip in U <sub>r</sub> ) for 25 cycles	
	<5 % U <sub>r</sub> (>95 % dip in U <sub>r</sub> ) for 5s	<5 % U <sub>r</sub> (>95 % dip in U <sub>r</sub> ) for 5s	
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE - U<sub>r</sub> is the a.c. mains voltage prior to application of the test level

## GUIDANCE AND MANUFACTURER'S DECLARATION ELECTROMAGNETIC IMMUNITY

The Free Action is intended for use in the electromagnetic environment specified below. The customer or the user of the Free Action should assure that it is used in such an environment.

IMMUNITY TEST	IEC 60601 TEST LEVEL	COMPLIANCE LEVEL	ELECTROMAGNETIC ENVIRONMENT – GUIDANCE
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3V	<p>Portable and mobile RF communications equipment should be used no closer to any part of the Free Action, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> $d = 1.17 \sqrt{P}$ <p>80 MHz to 800 MHz</p> $d = 2.33 \sqrt{P}$ <p>80 MHz to 2.5 GHz</p> <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation Distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range.</p>
Radiated RF IEC 61000-4-3	3V/m 80 MHz to 2.5 GHz	3V/m	<p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

**NOTE 1** At 80 MHz and 800 MHz, the higher frequency range applies.

**NOTE 2** These guidelines may not apply in all situations.

*Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.*

**a.** Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Free Action is used exceeds the applicable RF compliance level above, the Free Action should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Free Action.

**b.** Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

## RECOMMENDED SEPARATION DISTANCES BETWEEN PORTABLE AND MOBILE RF COMMUNICATIONS EQUIPMENT AND THE FREE ACTION

The Free Action is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Free Action can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Free Action as recommended below, according to the maximum output power of the communications equipment.

RATED MAXIMUM OUTPUT POWER OF TRANSMITTER (W)	SEPARATION DISTANCE ACCORDING TO FREQUENCY OF TRANSMITTER (M)		
	150 KHZ TO 80 MHz $d = 1.17 \sqrt{P}$	80 MHz TO 800 MHz $d = 1.17 \sqrt{P}$	800 MHz TO 2.5 GHz $d = 2.33 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.37	0.37	0.74
1	1.17	1.17	2.33
10	3.69	3.69	7.38
100	11.67	11.67	23.33

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

**NOTE 1** - At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

**NOTE 2** - These guidelines may not apply in all situations.

*Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.*

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In case of damage, please return appliance and warranty card to your retailer.

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